

# XSTAT<sup>®</sup> 12

## 3-PACK

NSN: 6510-01-657-4737

Part Number: FIN-0001-02

Each applicator contains  
approximately 38 minisponges.

**FOR USE BY TRAINED  
EMERGENCY RESPONDERS**

Go to [www.revmedx.com](http://www.revmedx.com)  
for training video and  
product information.

**ATTENTION: XSTAT 12 contains rapidly  
expanding cellulose sponges individually marked  
with an X-shaped radiopaque marker.**

### INDICATIONS FOR USE:

XSTAT 12 is a hemostatic device for the control of severe, life-threatening bleeding from junctional wounds in the groin or axilla not amenable to tourniquet application in adults and adolescents.

XSTAT 12 is a hemostatic device for the control of severe, life-threatening bleeding from narrow entrance extremity wounds in the arms or legs in adults or adolescents.

XSTAT 12 is a temporary device for use up to four (4) hours until surgical care is acquired. XSTAT 12 should only be used for patients at high risk for immediate life-threatening bleeding from, hemodynamically significant (Advanced Trauma Life Support class 3 or 4 hemorrhagic shock), non-compressible junctional wounds or narrow entrance extremity wounds, and when definitive care at an emergency care facility cannot be achieved within minutes.

XSTAT 12 is NOT indicated for use in: the thorax; the pleural cavity; the mediastinum; the abdomen; the retroperitoneal space; the sacral space; tissues above the inguinal ligament; or tissues above the clavicle.

### WARNINGS/CAUTIONS:

- XSTAT 12 has not been tested for use in extremity wounds that are amenable to tourniquet application.
- XSTAT 12 use in conjunction with tourniquet application has not been assessed for use in extremity wounds that are amenable to tourniquet application.
- Sterility not guaranteed if the package is damaged.
- Injuries with significant cavitation, such as those from a high-velocity gunshot wound, may require more than 3 applicators to appropriately pack the wound.

**TRAINING WEBSITE: [WWW.REVMEDX.COM](http://WWW.REVMEDX.COM)**

Label Part Number: RAW-0057-21

Label Issuance Date: 2017-08-15

Patent(s): [www.revmedx.com/patents](http://www.revmedx.com/patents)

**NOT MADE WITH NATURAL RUBBER LATEX  
STORE DEVICE IN ROOM TEMPERATURE**