

## 1-PACK

NSN 6510-01-644-7335 Part Number: FIN-0001-03

Quantity: 1 Applicator

Applicator contains approximately 92 minisponges.

# FOR USE BY TRAINED EMERGENCYRESPONDERS

Go to www.revmedx.com for training video and product information.

**ATTENTION:** XSTAT contains rapidly expanding cellulose sponges individually marked with an X-shaped radiopaque marker.

### **INDICATIONS FOR USE:**

XSTAT 30 is a hemostatic device for the control of severe, life-threatening bleeding from junctional wounds in the groin or axilla not amenable to tourniquet application in adults and adolescents.

XSTAT 30 is a hemostatic device for the control of severe, life-threatening bleeding from narrow entrance extremity wounds in the arms or legs in adults or adolescents.

XSTAT 30 is a temporary device for use up to four (4) hours until surgical care is acquired. XSTAT 30 should only be used for patients at high risk for immediate life-threatening bleeding from, hemodynamically significant (Advanced Trauma Life Support class 3 or 4 hemorrhagic shock), non-compressible junctional wounds or narrow entrance extremity wounds, and when definitive care at an emergency care facility cannot be achieved within minutes.

XSTAT 30 is NOT indicated for use in: the thorax; the pleural cavity; the mediastinum; the abdomen; the retroperitoneal space; the sacral space; tissues above the inquinal ligament; or tissues above the clavicle.

#### **WARNINGS/CAUTIONS:**

- XSTAT 30 has not been tested for use in extremity wounds that are amenable to tourniquet application.
- XSTAT 30 use in conjunction with tourniquet application has not been assessed for use in extremity wounds that are amenable to tourniquet application.
- Sterility not guaranteed if the package is damaged.
- Larger wounds may require more than one applicator. Having at least three (3) XSTAT 30 devices available at all points of care is recommended.

#### TRAINING WEBSITE: WWW.REVMEDX.COM

Label Part Number: RAW-0006-23 Label Issuance Date: 2017-08-15 Patent(s): www.revmedx.com/patents

NOT MADE WITH NATURAL RUBBER LATEX
STORE DEVICE IN ROOM TEMPERATURE