

# XSTAT<sup>®</sup>

**3-PACK**

**FOR USE BY  
TRAINED EMERGENCY  
RESPONDERS**

**NSN 6510-01-632-9440**

**Part Number: FIN-0001-01**

**Quantity: 3 Applicators**

**Each Applicator contains approximately 92 pellets**

**Go to [www.revmedx.com](http://www.revmedx.com) for  
training video and product  
information.**

US PATENT 8,828,050 B2

**ATTENTION: XSTAT contains rapidly expanding cellulose sponges coated with chitosan and individually marked with an X-shaped radiopaque marker.**

## INDICATIONS FOR USE:

XSTAT is a hemostatic device for the control of severe, life-threatening bleeding from junctional wounds in the groin or axilla not amenable to tourniquet application in adults and adolescents.

XSTAT is a temporary device for use up to four (4) hours until surgical care is acquired. XSTAT should only be used for patients at high risk for immediate life-threatening bleeding from, hemodynamically significant (Advanced Trauma Life Support class 3 or 4 hemorrhagic shock), non-compressible junctional wounds, and when definitive care at an emergency care facility cannot be achieved within minutes.

XSTAT is NOT indicated for use in: the thorax; the pleural cavity; the mediastinum; the abdomen; the retroperitoneal space; the sacral space above the inguinal ligament; or tissues above the clavicle.

## WARNINGS/CAUTIONS:

- XSTAT has not been tested for use in extremity wounds that are amenable to tourniquet application.
- XSTAT use in conjunction with tourniquet application has not been assessed for use in extremity wounds that are amenable to tourniquet application.
- Sterility not guaranteed if the package is damaged
- Contains material derived from shellfish.
- XSTAT elicited a mild pyrogenic response in biocompatibility tests. Monitor patient for rise in temperature, chills, hypotension, and septic shock.

**REVMED** X<sup>™</sup>

RevMedx, Inc.  
25999 SW Canyon Creek Road Suite C  
Wilsonville, OR 97070  
PH: 503.218.2172 [www.RevMedx.com](http://www.RevMedx.com)  
Label ID: RAW-0006-T1  
Label Issuance Date: 2015-12-09

STERILE R

**ONE TIME USE**

NOT MADE WITH NATURAL RUBBER LATEX  
STORE DEVICE IN ROOM TEMPERATURE

**RxOnly**