

ATTN TREATMENT FACILITY: A hemostatic dressing for non-compressible hemorrhage was used in the treatment of this casualty. Each device comprises approximately 92 small hemostatic sponges individually marked with an X-shaped radiopaque marker.

WARNING: Triangular segments of the applicator tip (FIG. 1) may break away during treatment and remain in the wound. These segments are visible under X-ray. If any separations occurred, the number of retained applicator tips may be noted on this casualty card.

FIG. 1: TRIANGULAR SEGMENT OF APPLICATOR TIF



## **PRODUCT REMOVAL INSTRUCTIONS:**


WARNING: Sponges must be removed intraoperatively by surgeon with the capability and equipment for achieving proximal and distal vascular control.

- Survey the wound site and assess potential vascular bleeding sites and develop plan to achieve surgical control of injured vessel(s).
- Remove sponges from the wound site manually and/or with surgical forceps to the site(s) of bleeding.
- Thoroughly explore wound and remove all sponges and any triangular segments of the applicator tip.
- 4) Prior to wound closure, obtain plane x-ray, optimally in more than one projection. The presence of retained sponges may be easily missed on radiographic images. Thoroughly examine x-ray for radiopaque x-pattern of sponges and any triangular segments of the applicator tip that may be inadvertently retained in the wound cavity.
- 5) If sponges or applicator tip segments are identified via x-ray, carefully re-examine wound cavity and remove them. Perform and review second x-ray to confirm complete sponge and applicator tip segment removal.
- 6) The XSTAT elicited a mild pyrogenic response in biocompatibility tests. Monitor patient for rise in temperature, chills, hypotension, and septic shock.

Contains material derived from shellfish.



RevMedx, Inc. 25999 SW Canyon Creek Road Suite C Wilsonville, OR 97070 PH: 503.218.2172 www.RevMedx.com Label ID: RAW-0019-01

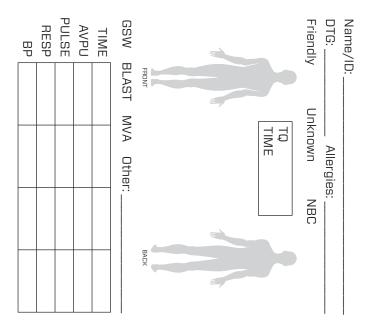

Medic'		Notes:	No. of )	Abx	Pain	DRUGS	Others:		FLUIDS:				œ	A:
Medic's Name:			No. of XSTAT applied:			DRUGS: (Type / Dose / Route):		He		Pressureux	J	ΩT	ChestSeal	Intact
			oplied:			/ Dose /		Hextend	NS/LR	reUx	J	Hemostatic	eal	Adjunct
			1 2			Route):		500	500	<		tatic	NeedleD	Inct
			ω					1000	1000	C		Packed		Cric
								Ö		XSIAI		ä	ChestTube	Intubated
									1500				Эe	ated