

PARABELT®

Tourniquet Belt

PARABELT is a daily-wear utility belt with an integrated ratcheting tourniquet. For use as a tourniquet, PARABELT is intended to be wrapped around a patient's limb and tightened to reduce circulation and therefore control severe bleeding of the arms or legs.



- Compact, Lightweight & Rugged
- Self-Locking System

INSTRUCTIONS FOR USE:



1. Place belt around injured limb 2-3 fingers above wound, with ratchet facing out. Thread belt through buckle over sliding bar.



2. Release Velcro and insert thumb into loop. Pull on free end to tighten belt securely around limb.



3. Lift lever arm of ratchet repeatedly to tighten tourniquet until bleeding stops. Record time of application. Seek immediate medical attention. Reassess the wound frequently: Additional tightening may be required.



4. Remove tourniquet by placing finger under tab marked "lift to open" and firmly pull upward and back to release. If tourniquet doesn't release, lift ratchet lever arm while pulling tab.

CARE & MAINTENANCE:

- Hand wash and air dry only.
- Keep tourniquet free of dirt and debris.

CE NOT MADE WITH NATURAL RUBBER LATEX
Assembled in the USA

REVMED X™

RevMedx, Inc.
25999 SW Canyon Creek Road Suite C Wilsonville, OR 97070
PH: 503.218.2172 www.RevMedx.com
© 2016 Revmedx, Inc.

EC REP FENTON PHARMACEUTICALS LTD, Fenton House, 4 Hampstead Gate,
1a Froggnal, London NW3 6AL, United Kingdom

Label ID: RAW-0066-01 Label Issuance Date: 2016-09-26 REF FIN-0008-MV

WARNING

TOURNIQUET IS INTENDED FOR USE ONLY IN LIFE THREATENING SITUATIONS TO CONTROL SEVERE BLEEDING OF THE ARMS OR LEGS.

Failure to read the instructions for use prior to using the tourniquet can result in misapplication and cause death, serious injury or loss of limb.

- READ instructions for use before using the tourniquet.
- Seek professional training before use if unfamiliar with the use of tourniquets
- KEEP these instructions and review them regularly (instructions can also be viewed at www.revmedx.com)
- Use tourniquet ONLY on arms and legs
- DO NOT place tourniquet directly on elbow or knee joint
- DO NOT apply tourniquet for more than 4 hours
- DO NOT place tourniquet around neck
- KEEP OUT OF REACH OF CHILDREN