



**Applicator Use:**

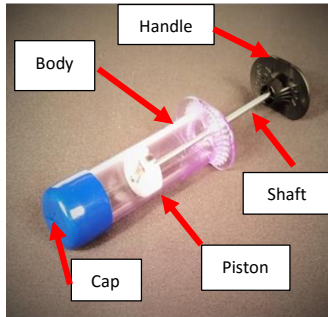

**STEP 1.**  
Hold the applicator in your hand or place on a hard surface with the white piston centered in the applicator body.



**STEP 2.**  
Insert a ball point pen (not included) through the blue applicator tip and place the tip of the pen through the round hole on the metal slide (see arrow above). **DO NOT** attempt to remove the blue tip.

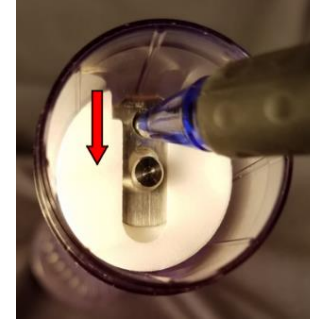

**STEP 3.**  
To release the piston from the shaft, press the metal slide towards the center of the white piston while applying light pressure on the handle. Piston will slide free when released.



**STEP 4.**  
Use the ball point pen to push the white piston down the shaft to the bottom of the applicator before refilling.



**STEP 5.**  
Hold applicator upright or place the applicator upright on a hard surface. Insert the funnel through the blue applicator tip and fill with sponge or pellets. **NOTE:** Angle shaft in a manner to allow sponges or pellets to pass through funnel. Manipulate applicator and/or sponges to expedite filling process.

**To Use:** Pull handle away from applicator body until it locks (listen for a “click”).  
Depress handle to expel sponges or pellets for training use.  
Discard expanded sponges after use. Pellets, funnel and applicator may be reused.



XSTAT Training Applicator and components are **NOT** Medical Devices.  
XSTAT Training Applicator and Components are **NOT** intended for human use.  
XSTAT Training Applicator and Components are **NOT** sterile.  
XSTAT Training Applicator components contain small parts - keep out of reach of children.

**XSTAT Training Mannequin**

The XSTAT Training Mannequin is intended for training in the use of XSTAT devices and other RevMedx trauma devices.



Extra reservoir cups (x3) and caps (x2)

Fluid Input Tubing

XSTAT 12 (Small Caliber) Wound Insert Cap and Reservoir

XSTAT 30 (Large Caliber) Wound Insert Cap and Reservoir

Extra wound reservoir and cap

Large wound Insert



The XSTAT 30 or Large Caliber Wound Insert is intended for use with the XSTAT 30 Training Applicator.

Apply lubricant to the interior of the wound model and to the applicator before each use.



The Large wound insert is intended for use to train on techniques such as wound packing with product such as XGAUZE®.

As needed, apply lubricant before each use with the wound packing products.



Training Fluids can be inserted using the attached tubing and the included syringe.



Training Fluids will be collected in the reservoir cup. The wound cap can be removed for cleaning or removal of the reservoir cup.