

# XSTAT<sup>®</sup> 30

## GENERATION 2

**ATTN TREATMENT FACILITY:**  
A hemostatic dressing for non-compressible hemorrhage was used in the treatment of this casualty. Each device comprises approximately 108 small hemostatic sponges individually marked with an X-shaped radiopaque marker.

### PRODUCT REMOVAL INSTRUCTIONS:

**WARNING:** Sponges must be removed intraoperatively by surgeon with the capability and equipment for achieving proximal and distal vascular control.

- 1) Survey the wound site and assess potential vascular bleeding sites and develop plan to achieve surgical control of injured vessel(s).
- 2) Remove sponges from the wound site manually and/or with surgical forceps to the site(s) of bleeding.
- 3) Thoroughly explore wound and remove all sponges.
- 4) Prior to wound closure, obtain plane X-ray, optimally in more than one projection. The presence of retained sponges may be easily missed on radiographic images. Thoroughly examine X-ray for radiopaque X-pattern of sponges that may be inadvertently retained in the wound cavity.
- 5) If sponges are identified via X-ray, carefully re-examine wound cavity and remove them. Perform and review second X-ray to confirm complete sponge removal.

### WARNING:

- 1) Relying upon minisponge count alone post removal is not an accurate means of determining complete sponge removal from the wound.
- 2) Careful surgical exploration of the wound site is required to ensure complete sponge removal from the wound.
- 3) Confirmation of complete removal from the wound by X-ray is required to search for possible retained minisponges.
- 4) Review of X-rays to identify potential retained XSTAT minisponges should be performed by physicians trained to review surgical X-rays.
- 5) While the minisponges are designed with an X-pattern radiopaque marker, it may be confused with other radiopaque material in the wound, such as bone chips and wound clips.



SEE PACKAGE INSERT  
FOR ADDITIONAL INFORMATION


**REVMED<sup>™</sup>**

RevMedx, Inc.  
25999 SW Canyon Creek Road Suite C  
Wilsonville, OR 97070  
PH: 503,218,2172 [www.RevMedx.com](http://www.RevMedx.com)  
Label ID: RAW-0102-01  
Label Date Issuance: 2018-06-18

Name/ID: \_\_\_\_\_

DTG: \_\_\_\_\_ Allergies: \_\_\_\_\_

Friendly      Unknown      NBC

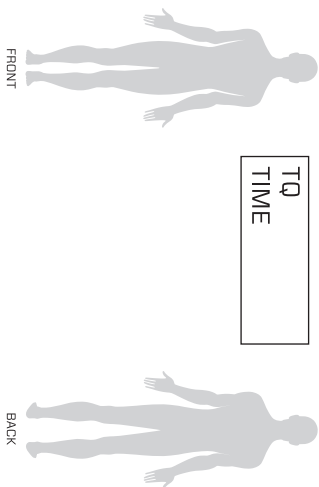

GSW    BLAST    MVA    Other: \_\_\_\_\_

|       |  |  |  |  |
|-------|--|--|--|--|
| TIME  |  |  |  |  |
| AVPU  |  |  |  |  |
| PULSE |  |  |  |  |
| RESP  |  |  |  |  |
| BP    |  |  |  |  |

A:    Intact    Adjunct    Cric    Intubated  
B:    ChestSeal    Needled    ChestTube  
C:    TQ    Hemostatic    Packed  
      PressuredX    IV    IO    XSTAT

|         |         |     |      |      |
|---------|---------|-----|------|------|
| FLUIDS: | NS / LR | 500 | 1000 | 1500 |
|         | Hexend  | 500 | 1000 |      |

Others: \_\_\_\_\_

DRUGS: (Type / Dose / Route):

Pain  
Abx

No. of XSTAT applied:    1    2    3

Notes: \_\_\_\_\_

\_\_\_\_\_  
\_\_\_\_\_  
\_\_\_\_\_  
\_\_\_\_\_  
\_\_\_\_\_  
Medic's Name: \_\_\_\_\_